



Millside House  
Anton Mill Road  
Anton Business Park  
Andover  
Hampshire, SP10 2RW  
t: +44 (0) 1256 896000  
e: [sean@poollifts.co.uk](mailto:sean@poollifts.co.uk)  
w: [www.poollifts.co.uk](http://www.poollifts.co.uk)

## MANAGEMENT OF MOBILE LIFTS FOR SWIMMING POOLS

The battery charge should be kept at a minimum of 50% charged as the lift is run on a trickle charge battery system. It is always beneficial to plug the machine into the mains when not in use.

The lift should be operated and tested at least twice a week to circulate the hydraulic oil through the main arm of the lift without a swimmer on board but with a light load on board. You may need to give a light push to the arm to get it moving without weight.

Always clean the lift after use by using clean fresh water and disinfecting solution on the seat.

Store the lift away from the swimming pool environment, ideally in a storeroom location away from areas of high humidity and chlorine.

It is advised that when the lift is not in use, remove the handset control to avoid non-authorised personnel operating the machine.

In the event of lift failure contact The Pool Lift Company immediately.